

# Bath Seats 6", 8" & 12"

90040, 90041 & 90042



---

**USER MANUAL**

---



**WARNING:** Please read this Manual carefully before using the **Bath Seats 6", 8" and 12"** as it contains important information regarding the safe use of the product.

# Contents

1.	Description of product .....	4
2.	Instructions of use .....	4
3.	Cleaning Instruction .....	6
4.	Technical Specifications .....	6
5.	Contact Us .....	8

## **1. Description of product**

This bath seat has been designed as versatile product to provide additional height when sitting in a bath and to provide added comfort to the user.

The bath seats are constructed from sturdy & lightweight PVC plastic.

The bottom section has four rubber feet to prevent any slippage of the seat while in use.

## **2. Instructions for use**

Place the bath seat in the required position in the bath. When removing the seat from the bath, first pull the tabs on the suckers to release the feet and then lift the seat out of the bath. Please store the seat safely.

Make sure that you have read the safety section before using the seat for the first time. The bath seat is available in three heights: 6", 8" & 12", each of these having 5 slats.

These seats are designed for a user to sit and be able to bathe, wash or shower in comfort. The seats have a Max. user weight limit of 190Kg (30stone)

Always consult your health care professional for advice on using the product safely.



**WARNING:**

- The bath seat is only designed for sitting upon. **DO NOT** stand or otherwise misuse the product.
- **Do not** exceed the stipulated Max. user weight on the seat.



**CAUTION:**

- Keep your hands and fingers clear of the gaps between the slats when using the seat.
- Ensure that the suckers on the feet of the seat are securely fixed to the bath to prevent any movement of the seat **before** each use.
- Keep this product out of direct sunlight or excessive heat, as this can cause the parts of the product to harden or crack.

**NOTE:**

- Some users may require supervision while using the seat. Please consult your healthcare professional for details.

### 3. Cleaning Instruction

Always remove the seat from the bath before cleaning. All parts should be cleaned periodically. Use a non-abrasive mild detergent along with warm water & a soft cloth to clean the seat. Rinse with clean warm water & a soft dry cloth. If disinfecting is required, a mild disinfectant that will not cause damage to plastics should be used.



#### CAUTION:

- Do not use any form of abrasive cleaner, cloth or sponge with this product as this could cause damage to the product. Strong acid, alkaline or products containing these chemicals may cause damage or discolouration to the product and should not be used.
- Autoclave cleaning is **NOT** recommended, as this can cause damage to the plastics used in the construction of the bath seat.

#### NOTE:

- After cleaning or disinfecting, all the parts should be showered down or wiped over with a damp cloth **before** the bath seat is used on the bath again.

## 4. Technical specification

<b>Product Code</b>	<b>Length</b>	<b>Width</b>	<b>Height</b>
90040	460mm	280mm	150mm
90041	460mm	280mm	200mm
90042	460mm	280mm	300mm

Maximum user weight limit of 190Kg (30stone)

## 5. Warranty

This product is guaranteed to be free of defects in materials or workmanship for one year from date of purchase.

This product has been built to high standards of workmanship and has been carefully inspected prior to shipment. This warranty is an expression of the confidence we have in the workmanship and materials used in our products. In the unlikely event of there being a defect found with the product, please contact your supplier and they will either replace the product or supply parts for the product to be repaired at their discretion.

This warranty does not cover failure of the device caused by misuse, negligence or fair wear and tear of the product.

## 6. Contact Us

Integrity Mobility Solutions customer service is available to answer questions from 8.00am to 4.30pm, Monday to Friday for business and upgrades. Our sales team is available for pricing and order processing and status queries. Our Technical Support team is available to answer your queries related to troubleshooting, please contact the following:

### **Integrity Mobility Solutions Limited**

Customer Service, Support, Sales Enquiries and Technical Support  
Thane House  
Hilmarton  
Wiltshire  
SN11 8SB

Tel: 01249 760582

Email: [sales@integritymobilitysolutions.com](mailto:sales@integritymobilitysolutions.com)



**Please Note:** Information in this user manual is subject to change without notice and does not represent commitment on the part of Integrity Mobility Solutions Limited. No part of this manual may be reproduced or transmitted in any form or by any means, electronic or mechanical, including photocopying and recording for any purpose without the written permission of Integrity Mobility Solutions.



---

Page intentionally left blank

---

---

Page intentionally left blank

---

---

Page intentionally left blank

---

Registered Office:  
74 St. Georges Road  
Bolton  
Greater Manchester  
BL1 2DD



Ives MD EU Rep. Ltd, 44 Rosses Stables,  
Ballymena, Co. Antrim, BT42 2SF

