

Height Adjustable Toilet Frame (Fixed Width) – 90028



Product Specification	
Maximum height	795 mm
Minimum Height	645 mm
Width	524 mm
Depth	450 - 455mm
Width Between Arms	430 mm
Weight	4Kg
Maximum User Weight	190kg (30st)

Use of the Product

This product has been designed to provide support in toileting to users who require help in standing or sitting. The frame is made from a rigid double strut steel tube that locates around the toilet. This item is designed to be removed from around the toilet. There are moulded plastic arms attached to provide a comfortable grip and easy cleaning.

Adjusting Height

The height of the frame can be adjusted by removing the e-clips on each of the legs & lifting (Or lowering) the frame to the desired height. Once the desired height has been achieved, replace the tube clips in the nearest hole in both tubes locking the frame in position. Ensure **all** e-clips are **fully inserted** through both tubes.

Cleaning

Use a non-abrasive mild detergent along with warm water & a soft cloth to clean the crutch. Rinse with clean warm water & a soft dry cloth. **DO NOT** use any form abrasive cleaner, cloth or sponge with this product as this could cause damage to the product. This product is suitable for thermal decontamination in an automated washing machine.

