

Fixed Arm Adjustable Height Commode - 90096

Assembly Instructions and User Guide

Thank you for choosing the Fixed Arm Adjustable Height Commode. This user guide should be read carefully before use as it contains important information about the product's safe operation and maintenance.



Product Details and Dimensions	
HEIGHT	785 – 930 mm
WIDTH	551 mm
DEPTH	520 – 545 mm
SEAT WIDTH	410 mm
SEAT DEPTH	410 mm
SEAT HEIGHT MIN	470 mm
SEAT HEIGHT MAX	620 mm
WIDTH BETWEEN ARMS	507 mm
WEIGHT CAPACITY	180 kg (28st)

WARNING: Max User Weight: 180kg (28st)

This product has been designed specifically to aid toileting requirements and consists of a coated tubular frame with a cushioned over seat and back rest, covered in a flame retardant, waterproof vinyl covering as well as a plastic hinged toilet seat and 5 litre pan and lid.

Assembly Instructions

- Carefully unpack and examine the product to ensure it is complete.
- Before use, check for any signs of damage e.g. cracked weld, flaking of paint, bent tube. If you spot any of these issues, please contact your supplier and refrain from using the product.

Do Not Attempt To Use If Product Is Damaged.

Operation Instructions

- Ensure legs of the commode are all set to the same height (see height adjustment)
- Please empty the potty at regular intervals and be aware of its maximum capacity of 5 litres.
- Do not attempt to move the commode whilst occupied or in use.
- To locate the potty in the commode, raise the toilet seat to a vertical position.
- By holding the sleeve or potty place on the support brackets ensuring an accurate location of the wide brim on the brackets.

- Remove lid (potty only) fold the potty handle down and lower the toilet seat into the horizontal position.
- Once the commode has been used, repeat the process in the reverse order ensuring that the lid (potty only) is correctly placed to avoid possible spillage.

Height Adjustment:

To adjust the height, remove the 'E' clip from the extension leg(s) and extend all legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg.

Once you have achieved the desired height; reinstall the 'E' clip, ensuring it has passed through the extension leg and that it fits neatly and snugly around the outside leg.

Cleaning

Prior to cleaning the chair, please remove all pin clips and the four extension legs with the ferrules fitted. Clean these items separately to the main frame.

To clean the product, please use a mild detergent with warm water, along with a soft cloth, to wipe down the framework on at least a weekly basis or more frequently if needed. If required, disinfect with chloride disinfectant, please ensure you rinse and dry.

The commode legs and retaining clips can be removed to aid cleaning. This feature is an added benefit over traditionally fixed commodes as it ensures all surfaces are fully accessible for cleaning.

Please do not use an abrasive cleaner as this may damage the frame and nullify the warranty. It is recommended that the commode potty is thoroughly cleaned after every use, using an appropriate cleaning agent and that the legs are removed from the frame, in order to clean.

Check

The product should be periodically checked for damage or wear and tear, this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to Integrity Mobility Solutions in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from

Warning

- This product is intended for users whose weight does not exceed 180 kg/28 stone.
- It is recommended that a full risk assessment should be carried out before use.
- Under no circumstance should the product be moved whilst in use.
- The product should not be "swung on" in a forward or backward motion which could render the product unstable and could cause folding inwards of front or back legs.
- It is advised that the frame is checked periodically for signs of wear and tear. If any problems are noted, please report to your local distributor immediately.