

Shower Stool with Handles – 90115

Assembly Instructions and User Guide

Thank you for choosing this Shower Stool with handles. This user guide should be read carefully before use as it contains important information about the products safe operation and maintenance.



Product Dimensions	
Overall Width	455 mm
Overall Depth	380 mm
Seat Width	380 mm
Seat Depth	280 mm
Seat Height	480 – 630 mm
Width Between Arms	485 mm
Maximum User Weight	160kg (25st)

WARNING: Max User Weight: 160kg (25st)

Use of the Shower Stool

- When moving always lift the shower stool by placing your hands on the tubular metal arms.
- Do not allow children or unauthorised persons to play with the equipment or use without proper supervision.
- Do not rock back and forth or sideways when using the stool as this will cause excessive stress and possible damage and failure which may cause injury.
- Do not use the shower stool outside the home or on uneven surfaces (such as in the garden) as this will cause stress to the legs and possible damage which may cause injury.
- Do not place fingers or foreign objects into any of the exposed tubular sections as this can lead to serious injury.

Cleaning

Prior to cleaning the stool please remove all pin clips and the four extension legs with the ferrules fitted. Clean these items separately to the main frame.

Clean the plastic seat using a non-abrasive soft cloth with a mild detergent. Abrasive cleaners and/ or abrasive cleaning pads could seriously damage the seat beyond repair.

Operation Instructions

To adjust the height, remove the 'E' clip from the extension leg and extend all legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg.

Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has passed through the extension leg and that it fits neatly and snugly around the outside leg.

Check

Periodically check to ensure all components are present, correctly assembled and suitably tightened.