

Toilet Frame & Clip-on Seat with Chute – 90210



Product Specification	
Maximum height	820 mm
Minimum Height	690 mm
Width	540 mm
Depth	370 mm
Width Between Arms	495 mm
Weight	4Kg
Maximum User Weight	190kg (30st)

Description

This product has been designed to allow the seat & chute to be easily removed to allow for easy cleaning of all parts, helping to minimise infection control issues. The frame is coated with a tough anti-scratch paint finish for improved longevity. It is also provided with a splash guard (Chute) and is designed to fit over the majority of standard toilet suites. The frame is height adjustable to suit a range of different users. The frame has integral arms to aid safe transfer. Non-slip ferrules are supplied as standard.

Adjusting Height

The height of the frame can be adjusted by removing the e-clips on each of the legs & lifting (Or lowering) the frame to the desired height. Once the desired height has been achieved, replace the tube clips in the nearest hole in both tubes locking the frame in position. Ensure **all** e-clips are **fully inserted** through both tubes.

Cleaning

Use a non-abrasive mild detergent along with warm water & a soft cloth to clean the crutch. Rinse with clean warm water & a soft dry cloth. **DO NOT** use any form abrasive cleaner, cloth or sponge with this product as this could cause damage to the product. This product is suitable for thermal decontamination in an automated washing machine.

Disposal

The frame is fully recyclable at any local recycling centre. The seat & chute are made from Polypropylene & as such may need to be disposed of separately. Please ask your local recycling centre for instructions on how to dispose of these items safely.