

**Slatted Bath  
Boards 26", 27" &  
28"  
90210, 90211 & 90212**



---

**USER MANUAL**

---



**WARNING:** Please read this Manual carefully before using the **Slatted Bath Boards 26", 27" and 28"** as it contains important information regarding the safe use of the product.

# Contents

1.	Description of product .....	4
2.	Instructions of use .....	4
3.	Cleaning Instruction .....	6
4.	Technical Specifications .....	7
5.	Contact Us .....	8

## **1. Description of product**

This bath board has been designed as a versatile product to provide additional height when sitting in a bath and to provide added comfort to the user.

The bath boards are constructed from sturdy & lightweight PVC plastic.

There are three different lengths (26", 27" & 28") which should be selected to match the overall width of your bath (Excluding the bath panel).

## **2. Instructions for use**

- Loosen the wing nuts on the underside of the board and slide both brackets towards the centre.
- Place the Bath Board on top of the bath, ensuring that the board sits evenly on both sides.
- Push each bracket towards the edge of the bath until they touch the inside edge of the bath.
- Secure the brackets by tightening the wing nuts until the board is unable to move.

Make sure that you have read the safety section before using the seat for the first time.

These boards are designed for a user to sit and be able to bathe, wash or shower in comfort. The boards have a maximum user weight limit of 190Kg (30 stone).



**WARNING:**

- The bath board is only designed for sitting upon. Do not stand or otherwise misuse the product.
- Do not exceed the stipulated maximum user weight for the board.



**CAUTION:**

- Ensure that the correct length of board for the bath is being used.
- Ensure that there is no more than 2.5cm (1") of overhang on the edge of the bath when installed.
- Ensure that the brackets on the underside of the board are preventing any movement of the board **before** each use. The rubber pads on the brackets **must** be touching the edge of the bath.
- Keep this product out of direct sunlight or excessive heat, as this can cause the parts of the product to harden or crack.

**NOTE:**

- Always consult your health care professional for advice on using the product safely and transferring either to or from the bath.

### 3. Cleaning Instruction

Always remove the board from the bath and remove the brackets before cleaning. All parts should be cleaned periodically. Use a non-abrasive mild detergent along with warm water & a soft cloth to clean the step. Rinse with clean warm water & a soft dry cloth. If disinfecting is required, a mild disinfectant that will not cause damage to plastics should be used.



#### CAUTION:

- Do not use any form of abrasive cleaner, cloth or sponge with this product as this could cause damage to the product. Strong acid, alkaline or products containing these chemicals may cause damage or discolouration to the product and should not be used.
- Autoclave cleaning is **NOT** recommended, as this can cause damage to the plastics used in the construction of the bath board.

#### NOTE:

- After cleaning or disinfecting, all the parts should be showered down or wiped over with a damp cloth **before** the bath board is used on the bath again.

## 4. Technical specification

Product Code	Width	Length	Height
90210 (4-slat)	225mm	660mm	110mm
90211 (4-slat)	225mm	685mm	110mm
90212(4-slat)	225mm	710mm	110mm

**Max user weight:** 190 kg | 30 Stone

## 5. Warranty

This product is guaranteed to be free of defects in materials or workmanship for one year from date of purchase.

This product has been built to high standards of workmanship and has been carefully inspected prior to shipment. This warranty is an expression of the confidence we have in the workmanship and materials used in our products. In the unlikely event of there being a defect found with the product, please contact your supplier and they will either replace the product or supply parts for the product to be repaired at their discretion.

This warranty does not cover failure of the device caused by misuse, negligence or fair wear and tear of the product.

## 6. Contact Us

customer service is available to answer questions from 8.00am to 4.30pm, Monday to Friday for business and upgrades. Our sales team is available for pricing and order processing and status queries. Our Technical Support team is available to answer your queries related to troubleshooting, please contact the following:



**Please Note:** Information in this user manual is subject to change without notice and does not represent commitment on the part of. No part of this manual may be reproduced or transmitted in any form or by any means, electronic or mechanical, including photocopying and recording for any purpose without the written permission of



---

Page intentionally left blank

---

---

Page intentionally left blank

---

---

Page intentionally left blank

---

