

# User Instructions

## 90290 – Overbed Adjustable Table

Maximum Weight Limit: 15Kg

### General Safety Warning

**Please ensure that these instructions are fully read and understood before installation / use. Retain these instructions in a safe place for future reference. Do not exceed the weight limit above – doing so could damage the equipment or put the user at risk.**

### Before Use

Carefully remove all packaging. Avoid use of knives or similar sharp instruments as these may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use the product, but contact Integrity Mobility Solutions for support.

### Contents:

- Wood finished table top
- Chrome plated “L” shape support tubes
- Painted tubular “H” section with black threaded locking nuts x 2
- U-shaped base with two short uprights
- E-Clips x 2

### Assembly

1. Carefully unpack the overbed adjustable Table and examine all parts for any damaged or loose fittings. Should you find any fault, please contact Integrity Mobility Solutions for an exchange.
2. After inspection, place the wooden finished table-top face down onto a soft surface to avoid any scratches and clip the chrome plate tubes into the clips on the underside of the table-top. Make sure both tubes are vertical and level.
3. Take the two painted tubular sections with the black threaded locking nuts attached & slide these over each of the chrome plated tubes, with the black locking nuts facing downwards toward the table-top. Tighten the nuts onto the chrome plated tubes.



4. The black locking nuts are designed to be tightened onto the chrome plated tubes, allowing the table-top to be positioned at any height or angle without the risk of collapse under its intended use. However, the design of this unit is only meant to be adjusted to a “Finger tight” pressure. Over tightening of the locking nuts could result in the plastic housing being damaged.
5. If already installed, remove the e-clips from the U-shaped base upright tubes.
6. Take the U-Shaped base and place the upright tubes inside the two tubular sections of the “H” section already attached to the chrome plated tubes.
7. Ensure that the pre-punched holes in both painted tubular sections line up with each other. (You may need to loosen the black locking nuts. If so, be sure to retighten the nuts once the alignment has been achieved.)
8. Insert the two e-clips into the aligned holes in the tubes, making sure that the e-clips are fully located through both tubes. **FAILURE TO DO SO WILL MAKE THE TABLE UNSTABLE.**

**N.B. Please note that over tightening of the nuts in this way will invalidate the warranty on this item, as this is misuse of the intended application.**

### **Care and Cleaning:**

Do not use abrasive cleaners with this product, as this could damage the surface of the equipment. For cleaning, wipe with a clean cloth soaked in detergent solution and wring out. Depending on the detergent manufacturer's instructions, the solution may be heated.

Wipe all surfaces of the equipment and the cloth should be re-immersed periodically and the detergent changed frequently. The detergent should be a clean cloth immersed in clean water and wrung out.

To disinfect, make sure that the equipment has been cleaned and dried before wiping over with a clean cloth and a compatible disinfectant. After disinfecting, clean with clean water and dry.

When using detergent or disinfectants, please refer to manufacturer's instructions. If you are unsure then consult your local Infection Control Specialist for advice.

The metal frame of this equipment can be cleaned / disinfected by moist heat at a temperature up to 80°C for a minimum of 1 minute or autoclaved at 80°C for at least 1 minute. After cleaning or disinfecting, treat all joints with a water dispersing agent (e.g. WD40).

**The wooden table top is not suitable for cleaning with moist heat or autoclaving.**

### **Height Adjustment**

To adjust the height of the chair to the most comfortable position, remove the four plastic clips that are found at the base of the legs. This will allow the leg extensions to drop down. Slide the leg extensions to the desired position, making sure that all leg extensions are at the same height.

Replace the plastic clips in the holes in the leg, securing the leg extensions in place, ensuring that each clip is located fully through each of the legs.

**DO NOT** exceed the maximum weight limit shown above.

Inspect at regular intervals for wear and damage.

### **LOCAL AUTHORITY AND HEALTH SERVICE**

**EMPLOYEES:** These instructions must be left with the table for the end user to refer to.

Manufactured in China and distributed by: