

Over-Bed Table with Castors – 90291

Assembly Instructions and User Guide

Thank you for choosing an over-bed Table. This user guide should be read carefully before you use the over-bed table, as it contains important information regarding the product, including the safe operation and maintenance of it.



Product Dimensions		
Width (mm)	Length (mm)	Height (From floor to top of table)
410	680	700mm – 1,000mm

Package Contents

Before attempting to assemble this product check that all parts are present and complete. If any parts are missing please contact your supplier.

Assembly

- 1) Push four castors into the holes in the underside of the frame base.
- 2) Remove the “E Clip” from each leg.
- 3) Place each leg over the base spigots.
- 4) Re-insert the “E Clips” to attach the legs to the spigots.
- 5) Unfold the support legs outwards. One support leg must be re-positioned before proceeding. Press the spring button and slide it under the clip.
- 6) Slacken off the hand wheels and insert the support legs into the legs on the base. Tighten the hand wheels to fix the table top at the required height and angle. Adjusting the angle when the table is in use the hand wheels only require slackening a turn or two they do not need removing.

Cleaning

To clean the table, use a soft and slightly moistened cloth with a mild detergent. It is not recommended to use any bleach, very hot water or any abrasive surface cleaners, as this may result in damage or mark the frame surface.

Operation Instructions

To adjust the height and/or angle of the table, first slacken the 2 short hand screws by turning anticlockwise. Adjust the table to suit, and secure by turning hand screws clockwise.

The table is not load bearing, and should not be used to support objects over 5kg (11lb). Please do not allow people to lean on the table.

Checks

Periodically check to ensure all components are present, correctly assembled and suitably tightened.