

Premier Bed Rail – 90292



Description

The Premier Bed Rail is the ideal solution for help in getting in and out of bed. It is functional, strong, hygienic and safe.

When not in use, it can be stored flat, then easily assembled and fitted to the bed in seconds without the need for any tools. It has been designed to work with both manual and electrically adjustable beds. It also fits conventional beds including divans, pine and steel or wooden framed domestic beds and is easily adjusted for maximum user comfort and security.

Specification


Hand Rail Height:	820mm
Hand Rail Width:	280mm
Floor Frame Length:	820mm
Floor Frame Width:	400mm
Min Clearance Under Bed:	30mm
Max. Clearance under Bed:	320mm
User Weight Limit:	160Kg (25St)

Cleaning


The Premier Bed Rail can be cleaned using any all-purpose household cleaner.

User Instructions

The Premier Bed rail is supplied in two sections; an upper half which comprises of the handle with movable plate which houses the locking hooks. The lower half is a footplate with two upright studs for locating the handle section onto. (See Pictures below)

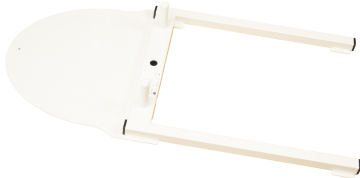

Footplate

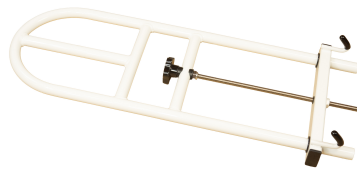

Handle

1. Remove the two sections of the product from the carton.
2. Place the footplate section of the unit onto the floor with the long legs facing the bed.
3. Place the handle section onto the two raised studs on the base plate, with the hooks facing the bed and insert the two e-clips supplied into the holes through the handle section and the studs. (See Fig. 1 Below)



Fig. 1

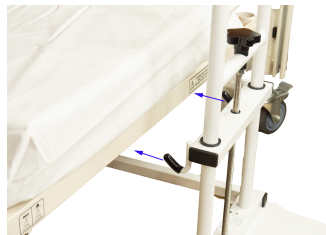


Fig. 2



Fig. 3

4. Move the assembled unit towards the bed until the locking hooks are beneath the edge of the bed. (See Fig. 2 above)
5. Rotate the Height Adjusting Handle clockwise to move the hooks upwards. (See Fig. 3 above)
6. Once the hooks are tight against the edge of the bed keeping the unit locked into position against the bed, the product is ready for use.

Maintenance

This product should be checked on a regular basis for any signs of damage that may affect the use of the product. If any signs of damage are found, then the user should not use the product with immediate effect. Please advise the supplier of the product of the damage found as quickly as possible, so that a replacement product can be provided.



Ives MD EU Rep. Ltd
44 Rosses Stables
Ballymena, Co. Antrim
N.I. BT42 2SF

