

Height Adjustable Toilet Frame with Injection Moulded Seat – 90341



Description

This Toilet Frame supports clients to give them confidence as they sit & rise, offering support and lessening of strain. It has an adjustable height feature to cater for varying patient requirements. The easy to clip on / clip off seat makes it easy to fit & remove and the partial cutaway provides easy access for personal cleansing. This product is suitable for intensive cleansing.

Specification

Toilet Seat Height:	438mm – 585mm
Width between Arms:	468mm
Toilet Seat depth:	419mm
Centre of Seat to top of Arms:	226mm
Footprint:	530mm – 585mm x 529mm
Aperture:	260mm x 230mm
Maximum weight of user:	190Kg (30 Stone)

Fitting Instructions

1. Carefully unpack and examine the product to ensure it is complete.
2. Before use, check for any signs of damage e.g. cracked weld, flaking paint, bent tube. If any signs occur, please contact your supplier to exchange and refrain from using the product.
Do not attempt to use if the product shows any sign of damage.
3. Remove the pin clips from the legs.
4. Select the desired height of the frame by gently sliding out each extension leg from the frame. NOTE: This is a telescopic function.
5. When the desired height is obtained, refit the pin clips into the appropriate hole ensuring that the pin clips go through each leg and its extension leg and that all legs are set to the same height.
6. Push the moulded seat onto the frame making sure that it is positioned as centrally as possible.
7. The frame is now ready for use.

Cleaning

Use mild detergent with warm water to wipe down the framework on at least a weekly basis or more frequently if required.

Do not use an abrasive cleaner as this will damage the frame.

Use of Power / Jet Washing and industrial Washing Machines

If either of the above are used to clean the equipment, care must be taken since high-pressure water can be penetrative.

Care should be taken that frames are not placed too close to high-pressure water jets.

After cleaning, ensure that any residual water is removed from the framework as it could cause permanent rusting of the frame. (Leave frames open in a warm environment for several hours to ensure that all internal moisture evaporates.)

We cannot offer advice on industrial washing machines with regards to specific machine settings as these will be dependent on the manufacturer's instructions.

Maintenance

The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking of paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the equipment until advice has been sought from the supplier.