



**90392 Teak Bath
Stool**

The luxury teak shower/bath stool has adjustable legs and rubber bushings and helps you bathe and shower more easily. It has been specially designed for use in the shower or bath.

If you have pain or weakness in your legs, back problems, can't bend over to reach your feet, or maybe you have balance problems or dizziness, then you should use our exercise stools/ luxury bath/shower.

In the bath, this means you don't have to lower yourself to the bottom of the bath - with the resulting problem of not being able to get out. In the shower, you can sit back and enjoy the water flowing over you without worrying about falling. Sitting on the stool also makes it easier to wash, especially the legs and feet.

It is height adjustable from 15 to 20 inches tall with push-button locking pins on each leg. It only weighs 7 pounds.

Like all of our products, the Luxury Teak Bath/Shower Stool is of the highest quality and comes with a money back guarantee if you are not completely satisfied.

- Attractive, high-quality teak is renowned for its durability and weather-resistant properties; Teak will naturally take on a beautiful silvery gray patina after several months of exposure to water
- Ideal for use in showers and large baths, also ideal for use as a ladies toilet stool
- It is height adjustable from 15 to 20 inches tall with push-button locking pins on each leg. It only weighs 7 pounds. It's a maximum of 24" wide
- Can take up to 21 stones / 300 kg
- Large rubber tips reduce slipping
- Easy, tool-free assembly.

Assembly Instructions

1. Place the first part of the support structure under the seat with the concave side upwards. Then place the second support piece on the convex side, over the first piece.

(See Figure 1)

2. Make sure that the holes provided for inserting the screws are aligned. Place the first butterfly nut screw in the center to secure the two parts of the support structure together.

(See Figure 2)

3. Screw the other wing nut screws through the pipes and into the benches. Make sure all nuts are tightened tightly.

(See Figure 3)

4. Slide the adjustable legs into each of the tubes of the support structure. Adjust to desired height. Make sure that the push button comes out completely in order to properly block the movement of the tab. Make sure all legs are at the same height.

(See Figure 4)



Contents:	Qty
Teak bath bench	1
Support structure	2
Adjustable legs	4
Wing nut screw	5

Limited lifetime warranty

Your branded product is guaranteed to be free from defects in materials and workmanship for the life of the original purchaser.

This device has been built to exacting standards and carefully inspected before shipment.

This limited lifetime warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of reliable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover appliance failure due to misuse or owner negligence, or normal wear and tear.

The warranty does not extend to non-durable components, such as rubber accessories, casters and handles, which are subject to normal wear and tear and must be replaced periodically.

If you have a question regarding your device or this warranty, please contact us.

