

ROTATING BATH SEAT

REF. 90398



USER MANUAL

ASSEMBLY INSTRUCTIONS



1. Remove contents from carton
2. Assemble legs by inserting the height adjustment legs into the white rubber tip. Make sure to put the tips on the end without the brass push pin. Repeat for 3 other legs.
3. Insert the height adjustment leg into larger leg receptable by depressing the brass push pin. Repeat for 3 other legs.
4. Lay seat upside-down on the flat surface. Insert each leg into the seat depressing the brass push pin on the leg receptable and inserting into the bottom of the seat. A click can heard when the leg is securely in place. Repeat for 3 other legs.
5. Turn the bath bench right-side-up. Attach the soap tray by placing it between the legs and inserting the cross braces into the receptables on the white leg collars.

ADJUSTING LEG HEIGHT

1. Before use, adjust height of each leg to your specific needs, making sure all legs are the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes.
2. To be sure legs are locked into place, push down on or pull up the legs.

ROTATING SEAT

1. To rotate seat, pull up on handle until seat is free of one of the 4 locked positions and rotate to desired position. Seat can rotate in both directions

Note : Built into the seat frame are 4 locking positions which will secure the seat in place to allow the user to safely stand up from or sit down on the seat. Make sure the seat is in one of the locked positions before standing or sitting.

REPLACING SEAT

The rotating seat and the two ball bearing rings that move the seat can be replaced if necessary. To remove seat and ball bearing rings, turn the lock knob (RTL12061-KNOB) under the seat (RTL 12061 SEAT) and ball bearing rings (RTL12061-RING). Replace seat, ball bearing rings and lock knob. Tighten until secure.

ADDITIONAL PRECAUTIONS

1. Inspect all parts for shipping, damage before assembly. If there is shipping damage. DO NOT USE. Contact dealer/carrier for further instructions.
2. Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position an even height is achieved.
3. Check the rubber tips on the leg extensions for rips wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
4. All leg extensions with rubber tips MUST touch the floor simultaneously at all times.
5. DO NOT Stand on chair.

Note : To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

WARRANTY

Your Brand Product is warranted to be free of defects in materials and workmanship for two years.

This warranty does not cover device failure due to owner misuse or negligence or normal wear and tear. The warranty does not extend to non durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement. If you have a question about your Integrity device or this warranty, please contact

TECHNICAL SPECIFICATIONS

Dimensions seat Diameter	36 cm.
Floor space	38 x 38 cm.
Total height	41/ 58 cm.
Weight bathing Seat	2.4 kg
Maximum supported weight	135 kg.
Cushion thickness	2 cm
Rotating seat: 360° with 90° lock	
Seat material: Polyurethane	
Non-slip feet	
Height-adjustable legs	17.5cm in 2.5cm steps



Maximum supported Weight

