

**Bariatric Commode Stool with PU Seat, PU back rest and Steel Arms -
90452**



Assembly Instructions and User Guide


Thank you for choosing this Commode stool with PU seat and back and steel arms. This user guide should be read carefully before use as it contains important information about the products safe operation and maintenance.

No tools are required for assembly

Product Details and Dimensions	
Width	663 mm
Depth	623-649 mm
Seat Width	628 mm
Seat Depth	489mm
Seat Height	434-584 mm
Width between Arms	616 mm
Height of Arms from Seat	180 mm
Maximum User Weight	280 kg (44st)

WARNING: Max User Weight: 280kg (44st)

Step 1

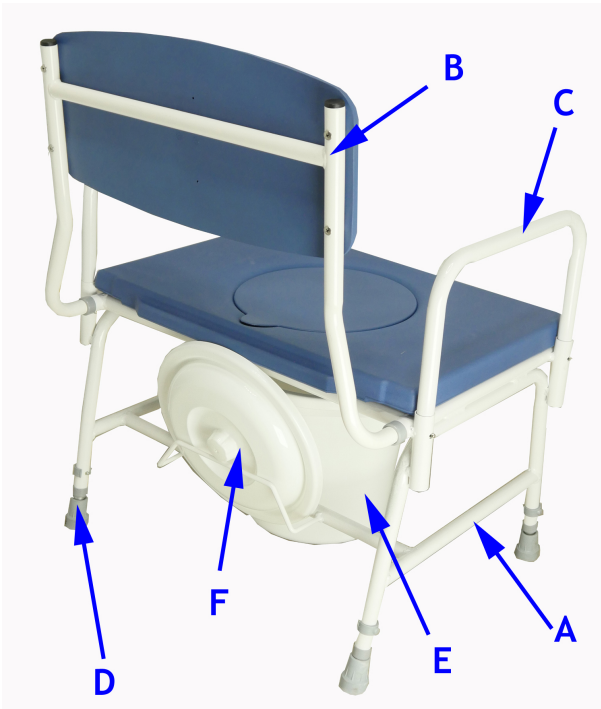

Check that all parts are present.

- A) 1 x PU seat and seat frame
- B) 1 x Backrest c/w PU back pad
- C) 1 x pair of arm rests
- D) 4 x Adjustable height legs
- E) 1 x Large Potty
- F) 1 x Potty Lid

STEP 2



Place the seat & frame on a flat surface with the two tubes facing backwards.

Take the backrest & place the tubes into the tube holes in the seat frame, pushing each tube in firmly **ensuring that the spring pins click into the holes securely.**

STEP 3



Fit the arms into each side of the stool. Press the spring pin in at each end & push the tubes into the sockets on the frame. Ensure spring pins fit securely as with the back rest.

To remove the arms, press the spring pin inside the tube & gently lift the armrest until the spring pin disappears. Do the same at the other end & then pull the armrest out of the sockets.

STEP 4



Insert one of the legs into one of the tubes on the lower section of the frame.

When the leg is seated properly the holes will line up on both tubes. Once you have the height you require, push the E-Clip through both tubes, locking the leg tube in place.

Fit the other three legs ensuring that they are all set at the same height and stand the stool upright

STEP 5



Push the potty under the chair from the front. There are some guide bars fitted to the frame that will fit underneath the rim of the potty. Push the potty as far back as the guide bars will allow. Place the lid of the potty into the rack that is fitted to the back of the chair.

Use of the Perching Stool

- When moving always lift the perching stool by placing your hands on the tubular metal frame beneath the seat.
- Do not allow children or unauthorised persons to play with the equipment or use without proper supervision.
- Do not rock back and forth or sideways when using the stool as this will cause excessive stress and possible damage and failure which may cause injury.
- Do not use the perching stool outside the home or on uneven surfaces (such as in the garden) as this will cause stress to the legs and possible damage which may cause injury.
- Do not place fingers or foreign objects into any of the exposed tubular sections as this can lead to serious injury.

Cleaning

Prior to cleaning the chair please remove both arms, backrest and all four extension legs with the ferrules fitted. Clean these items separately to the main frame.

Clean the upholstery using a non-abrasive soft cloth with a mild detergent. Abrasive cleaners and/ or abrasive cleaning pads will seriously damage the upholstery beyond repair.

Check

Periodically check to ensure all components are present, correctly assembled and suitably tightened.