

## T-Framed Footstool C/W Ferrules - 90454



### **DESCRIPTION**

Adjustable Leg rests with contoured calf support. Upholstery fire retardant to IGM source 5. E-clips allow the units to be adjusted for both height and angle.

The Height Adjustable Static Leg rest is fitted with non-slip rubber feet.

### **TECHNICAL SPECIFICATIONS**

Top size: 28cm x 50cm (11" x 19.7")

Weight limit: 100kg (16st)

Floor to Top Height 37CM (14 ½") – 52CM (20 ½")

### **USAGE REQUIREMENTS**

1. Do not exceed user weight of 100kg (16 Stone)
2. Ensure the e-clips are securely located into holes on frame when adjusting.
3. The frames are not designed to be sat upon.

## **CLEANING**

This product should be cleaned with a damp cloth & dried using a clean dry cloth. Do not use any abrasive cleaners or pads on the product as this could damage the surface of the leg rest pad or the frame.

## **DISPOSAL**

This product is suitable for recycling at a local recycling centre. The main frame is produced from Mild Steel tubes which have been powder coated. The ferrules are produced from TPR (Thermo-plastic rubber) & do not contain any latex. The footrest is made up of PVC, Fire retardant foam & wood. Please discuss with your local recycling centre about the suitability of recycling these parts of the product.

## **WARRANTY**

This product is supplied with a two year warranty, which covers any manufacturing faults that may appear within the warranty period, but does not cover normal wear & tear or any misuse of the product.