

Mobile T-Framed Footstool - 90455



DESCRIPTION

Adjustable Leg rests with contoured calf support. Upholstery fire retardant to IGM source 5. E-clips allow the units to be adjusted for both height and angle.

The Height Adjustable Mobile Leg rest is fitted with braked castors.

TECHNICAL SPECIFICATIONS

Top size: 28cm x 50cm (11" x 19.7")

Weight limit: 100kg (16st)

Floor to Top Height 37CM (14 ½") – 52CM (20 ½")

USAGE REQUIREMENTS

1. Do not exceed user weight of 100kg (16 Stone)
2. Ensure the e-clips are securely located into holes on frame when adjusting.
3. The frames are not designed to be sat upon.

CLEANING

This product should be cleaned with a damp cloth & dried using a clean dry cloth. Do not use any abrasive cleaners or pads on the product as this could damage the surface of the leg rest pad or the frame.

DISPOSAL

This product is suitable for recycling at a local recycling centre. The main frame is produced from Mild Steel tubes which have been powder coated. The ferrules are produced from TPR (Thermo-plastic rubber) & do not contain any latex. The footrest is made up of PVC, Fire retardant foam & wood. Please discuss with your local recycling centre about the suitability of recycling these parts of the product.

WARRANTY

This product is supplied with a two year warranty, which covers any manufacturing faults that may appear within the warranty period, but does not cover normal wear & tear or any misuse of the product.