

# KD Perching Stool with PU Seat, PU back rest and Steel Arms - 90626



## Assembly Instructions and User Guide

Thank you for choosing this Perching stool with PU seat and back and steel arms. This user guide should be read carefully before use as it contains important information about the products safe operation and maintenance.

### No tools are required for assembly

Product Details and Dimensions	
Width	560-635 mm
Depth	450-460 mm
Seat Width	325 mm
Seat Depth	265 mm
Seat Height	545-685 mm
Width between Arms	505 mm
Height of Arms from Seat	175 mm
Maximum User Weight	160 kg (25st)

**WARNING: Max User Weight: 160kg (25st)**

## Step 1



Check that all parts are present.

- A) 1 x PU seat and seat frame
- B) 1 x Backrest c/w PU back pad
- C) 1 x pair of arm rests
- D) 4 x Adjustable height legs

## STEP 2



Place the seat & frame on a flat surface with the two tubes facing upwards.

Take the backrest & place the tubes into the tube holes in the seat frame, pushing each tube in firmly **ensuring that the spring pins click into the holes securely.**

### STEP 3



Fit the arms into each side of the stool. Ensure spring pins fit securely as with the back rest (Notice they have a distinctive shape)

The **RED** arrow shows the front and the **BLUE** shows the back.



### STEP 4



Insert one of the legs into one of the tubes on the lower section of the frame.

When the leg is seated properly the button will pop up into the hole and lock it in place

Fit the other three legs and stand the stool upright



## STEP 5



desired height. Ensure that the stool is stable and at the desired height before use.

**Note the perching stool is designed to slant to the front. Do not adjust any of the legs to make the seat level as this will cause excessive stress to the legs compromising user safety and possibly leading to permanent damage of the perching stool, including fracture of the tubular and other sections.**

### Use of the Perching Stool

- When moving always lift the perching stool by placing your hands on the tubular metal frame beneath the seat.
- Do not allow children or unauthorised persons to play with the equipment or use without proper supervision.
- Do not rock back and forth or sideways when using the stool as this will cause excessive stress and possible damage and failure which may cause injury.
- Do not use the perching stool outside the home or on uneven surfaces (such as in the garden) as this will cause stress to the legs and possible damage which may cause injury.
- Do not place fingers or foreign objects into any of the exposed tubular sections as this can lead to serious injury.

### Cleaning

Prior to cleaning the chair please remove both arms, backrest and all four extension legs with the ferrules fitted. Clean these items separately to the main frame.

Clean the upholstery using a non-abrasive soft cloth with a mild detergent. Abrasive cleaners and/ or abrasive cleaning pads will seriously damage the upholstery beyond repair.

### Check

Periodically check to ensure all components are present, correctly assembled and suitably tightened.

### Warranty

Your Integrity product is warranted to be free of defects in materials and workmanship for one year from date of purchase. This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer. In the event of a defect covered by this warranty, we will at our option supply parts or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.