
Removal of the armrest and backrest: Two people are needed. One person presses down the armrest (1) and the pinprotrudingheadsatthelowerpartofthebackrest (2) with both hands, and the other person holds and lift the armrest (1) and backrest (2) lightly to remove them.

VI. Maintenance

Symbol and signification

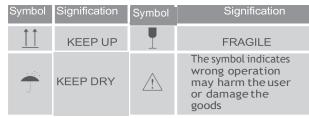

- Before use, please check whether all fittings of the product are fastened firmly and make sure there is no loose- ness.
 It is recommended to have a guardian when using the commode chair.
- When adjusting the height, make sure that the height of the four feet is the same, and the positioning pins
- (10) are fastened firmly.
 When installing the armrest (1), backrest (2) and upper

- leg tubes (9), make sure there is a "click" sound and check whether they are fastened firmly, so as to ensure safety. The foot pads (14) of the commode chair are made of
- plastic, and should be replaced intimeif anywear to avoid accidents.
- Attention: People with lower limbs inconvenience must pay attentiontosafetywhenusingthisproduct.
- Transportation: It should be handled with care during transportation.
- Donot throw, pressit with heavy load or rollitover. Storage: It should be placed in a dry, ventilated and stable room with
- ▶ nocorrosive materials.
- Use a soft dry cloth for wiping the medical aid.
 Discard the waste in accordance with the relevant local
 environmental protection regulations.

▶ All specifications and product configurations are subject to change without notification.

06

Æ

05



90667 Commode Chair User's Manual

Please read the user's manual closely before using!

Product usage

- ▶ It is used to assist patients with lower limb walking inconvenience after surgery and patients with lower limb disabilities to walk or stand for rehabilitation training.
- ▶ Contraindications: None

II. Main properties


Commode chair: removable armrest and backrest, and heightadjustableinsixlevels.

III. Model description

Commode chair has a white main body and adopts foot pads.

IV. Basic structure

It is composed of a commode rack, telescopic tubes, an armrest, an armrest tube, a backrest tube, positioning pins, foot pads, a commode frame, a commode lid, a commode bucket, and a commode bucket lid.
 The basic structure is asshown in the figure:

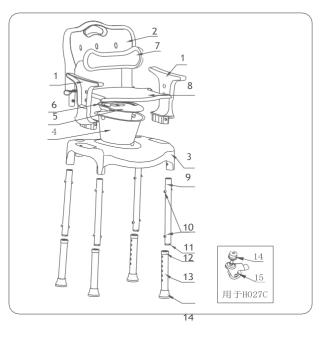

- 1. Armrest (including the structure functions of armrest and armrest tube)
- 2. Backrest(includingthestructurefunctionsofbackrest and backrest tube)
- 3. Chair seat (including the structure functions of commode rack and commode frame)
- 4. Commode bucket
- 5. Commode bucket lid
- 6. Commode bucket handle

- 7. Backrest cushion
- 8. Seatcushion (including the structure function of commode lid)
- 9. Upper leg tube
- 10. Positioning pin
- 11. Innertube plug
- 12. Pipe sleeve
- 13. Telescopic tube
- 14. Foot pads

V. Usage

 Selection of medical walking aids: The commode chair has removable armrests and back-rests, and the height is adjustable among six levels;

Height adjustment: Press the protruding head of the positioning pin (10) with one hand, and hold the telescopic tube (13) with the other hand, move it up and down to adjust to a suitable position, and ensure the protruding heads of positioning pins (10) of the 4 feet are

fastened firmly to the holes of the same height to avoid danger. ► Installation and use of commode chair:

• Insert 4 upper legtubes (9) into the foot holes of the seat (3) until hear a "click" sound. When the protruding head of the pin (10) pops out of the positioning hole, it means the leg tubes are installed.

• Insert the armrest (1) and the backrest (2) into the fixing holesoftheseat (3) in the same way of ① until hear a "click" sound. When the protruding head of the pinsingle pinpops out of the positioning hole, it means the leg tubes are installed.

 \cdot Place the commode bucket components (4)(5)(6) into the middle hole of the seat (3), and then put on the seat

cushion (8) in place.

• Clip the backrest cushion (7) into the 3 small oval holes on the backrest (2). Now the installation is complete and it can be used normally.

• When using the commode function, please remove the seat cushion (8) and the commodelid (5), and cover it again after using to prevent the bads mell of stool and urine; when using the bath function it is recommended to remove the commode bucket components (4)(5)(6).