

Scorpio Bed Back Support & Headrest – 90668



Thank you for choosing this Scorpio Bed Back Support & Head rest. This user guide should be read carefully before use as it contains important information about the safe operation and maintenance of this product.


This product is only designed for use by one person. There are a number of slots to provide different angle settings for the comfort of the user. The Backrest section folds flat for storage.

WARNING: Max User

Weight: 127kg (20st)

Use of the Bed Back Support

- Lay the unit out flat on top of the mattress & bedsheets.
- Lift the backrest up letting the cross bar drop down behind.
- Locate the Cross Bar into the two grooves that provide the closest angle to the desired position of the user.
- The headrest is held onto the backrest section by velcro. If you wish to remove the headrest from being used, lift it from the backrest until the velcro pulls apart. The headrest can now be placed behind the backrest. Please be aware that the headrest is permanently attached by fabric to the top tube. **DO NOT** attempt to remove the headrest from the top tube.

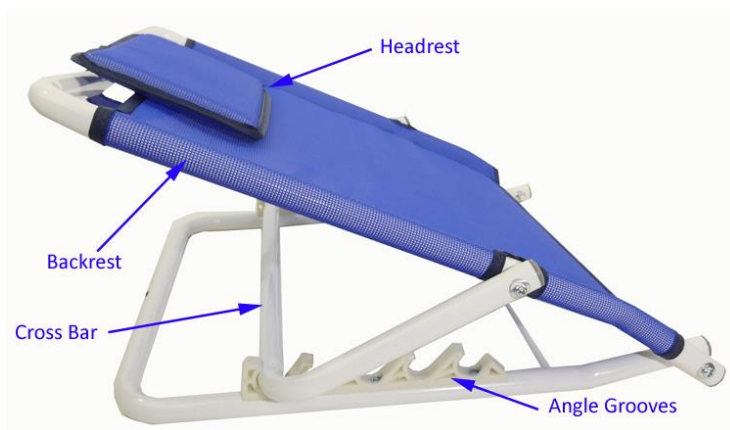

Folding of the Back Support

To fold the Back Support, pull the Backrest tube upwards & fold the Cross Bar under the Backrest, allowing the Backrest to drop down flat onto the bed sheet below.

Cleaning

The Bed Back Support & Headrest can be cleaned using a non-abrasive soft cloth with a mild detergent. Abrasive cleaners and/ or abrasive cleaning pads will seriously damage the upholstery beyond repair.

Warranty

The product comes with a 12-month warranty from date of purchase & does not cover the upholstery or normal wear & tear. Please ask your supplier for details of the warranty.

Check

Periodically check to ensure all components are present, correctly assembled and suitably tightened.