

Raised Toilet Seat

User Instructions for the following

Part Numbers: 90635, 36 & 37

2", 4" & 6"



Please ensure these instructions are fully read and understood before installation / use. Retain these instructions in a safe place for future reference.

Intended Use

The Raised Toilet Seat is fitted to a standard toilet bowl and is designed to provide a secure raised toilet seat surface for those with reduced mobility. It is available in three height options (2", 4" & 6") & either with or without a lid.

Contents

Raised Toilet Seat (With or without lid)

Two-Part Side Brackets (x2)

Check all parts carefully before assembling the product. Do not assemble if any of the parts are damaged. Do not assemble if any of the contents are missing.

Fitting Instructions


1. Assemble the side brackets; Place screw into threaded section of "L" shaped bracket as shown in fig 1.
2. Slot bracket into threaded slots in underside of seat as shown in fig 2 and secure using screw.

2 - Fitting to the Toilet

(Note: Ensure the toilet to which the seat is being fitted is safe and secure.)

3 - Lift the existing toilet seat up.

4 - Loosen the two side brackets an equal amount by turning the screws anti-clockwise and position the seat on the toilet bowl, ensure the front lip is outside of the bowl and the seat is evenly positioned about the bowl (see main picture)

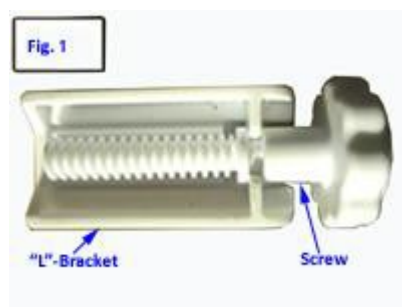

5 - To ensure the toilet seat is fitted correctly to the bowl, tighten each side bracket by turning both screws simultaneously, an equal amount, until the seat is secure.

6 - To remove the seat, simply unscrew the side brackets an equal amount and lift clear.

Safe Use of Toilet Seat:

- We recommend that you regularly check your seat, ensuring that the screws are tight and the seat remains evenly positioned on the bowl.
- If you suspect that the seat has moved out of position, please follow the instructions above to move it back into the correct position and retighten all fixings.
- Do not under any circumstances stand on the seat or allow others to do so as this is highly dangerous and will compromise the safety of the user.
- If you have to remove the Raised Toilet Seat for any reason, please ensure that you follow the installation instructions again fully before using. Failure to do so could compromise the safety of the user.

Safety Precautions

- Do not exceed the maximum user weight limit of 30st (190kg)
- Inspect the Toilet Seat regularly to ensure it is fitted correctly, is safe and secure, and fit for use. If in any doubt, contact a competent installer of sanitary equipment.
- Inspect at regular intervals for wear and damage.
- Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

Cleaning Instructions

- Clean regularly, use a general purpose, neutral detergent and warm water
- Please check the suitability of the product before use. Do not use abrasive cleaners as this will permanently damage the surface and invalidate any warranty.
- Do not use any form of solvent as this will damage the surface and weaken the product and again invalidate any warranty.
- To disinfect, wipe with a mild chloride disinfectant (solution of 1ml per litre).
- These seats may be autoclaved on cleaning cycles up to 73°C
- Dry the seat thoroughly before use.

Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non hazardous and can be disposed of appropriately.

